
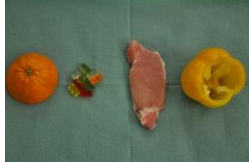
















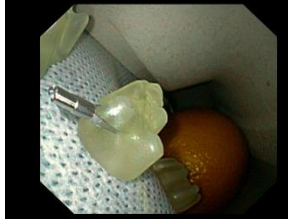








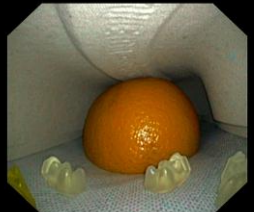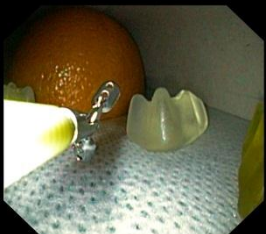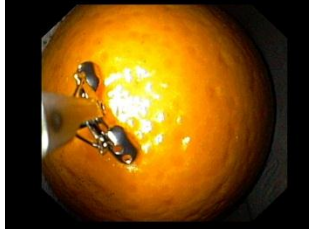





Appendix 6 Plan: How to make “home made dummies”

Preparations		Description
		<p>Equipment needed:</p> <ul style="list-style-type: none"> Gloves Table with water resistant covering Kidney basin, made out of paper / board Mouth guard Scalpel / Scissor adhesive film Neutral plates (when using HF-surgery) Oranges, wine gums, Pepper, Meat (2cm thick)
		<ul style="list-style-type: none"> Make a hole in the kidney basin In the size of the mouthguard Fix the mouthguard in the hole
		<ul style="list-style-type: none"> mouthguard protects the endoscope from any damages
		<ul style="list-style-type: none"> Measure the size of film to close the dummy
		<ul style="list-style-type: none"> Peel away / remove protective film
Dummy for Biopsy, injection and foreign body removal		
		<ul style="list-style-type: none"> Fix half of an orange or wine gums on the adhesive film put the kidney basin over the film
		<ul style="list-style-type: none"> close the basin by fixing the film on the side
		<ul style="list-style-type: none"> Insert an Endoscope via the mouthguard

		<ul style="list-style-type: none"> •
<p>Example for biopsy</p>		
		<ul style="list-style-type: none"> • Wine gums and oranges can be used for taking biopsies
<p>Example for injection techniques</p>		
		<ul style="list-style-type: none"> • Injection into wine gum • Simple ink, indigo carmine or methylene blue make the injection more visible
<p>•</p>		
<p>Example for Clipping</p>		
		<ul style="list-style-type: none"> • Wine gums can be used for injection techniques and clipping
<p>Example for polypectomy</p>		
		<ul style="list-style-type: none"> • Wine gum can be used for polypectomy
<p>•</p>		

Dummy for Polypecty, Ligation		
		<ul style="list-style-type: none"> • Make a hole in the meat
		<ul style="list-style-type: none"> • Fix the meat at the mouthguard
		<ul style="list-style-type: none"> • Fix the meat inside the kidney basin with an adhesive film • And close the whole basin with another film
		<ul style="list-style-type: none"> •
		<ul style="list-style-type: none"> • Insert the endoscope via the mouthguard and use air insufflation to blow up the meat inside the basin
		<ul style="list-style-type: none"> •
ERCP Dummy		
		<ul style="list-style-type: none"> • Tubes in different sizes are used for the ERCP Dummy • The larger tubes are used in anesthesiology • The Y-part is used in thoracic drainages • Cut a lateral hole in the bigger tube • Fix the thinner tube in the hole • Use a paper handkerchief as a papilla • Fix the handkerchief on the connector
		<ul style="list-style-type: none"> • Example of a plastic stenting